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香港區 Hong Kong

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香港 Hong Kong

以下商戶會售賣奶泵配件 Also sell breast pump accessories

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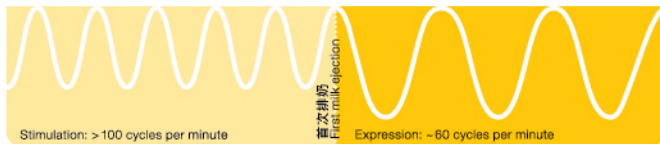
Medela understands that mothers sometimes need a little gentle support with breastfeeding. To cope with mothers' individual needs, for more than 50 years Medela constantly works with universities and dedicates to develop a collection of breastfeeding products, from breastpumps, breastcare products, breastmilk storage to a full set of accessories. **All Medela products are Bisphenol-A free, and are totally safe for babies.**

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2 Phase Expression is the breast pump technology developed by Medela. It mimics the infant's natural sucking behavior.

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Stimulation Phase

先模仿嬰兒吸嘔節奏，以較輕力度和較快速度按摩乳房兩分鐘*，每分鐘多於100次，刺激噴奶反射。
Starts with a rapid and light suction to massage the breast for 2 minutes* (>100 cycles / minute) which stimulates the milk ejection reflex (MER) and induce milk (let down).

*母親也可因應個人需要提早轉至「擠奶模式」
Can be switched to "Expression Phase" earlier depending on mother's individual needs.

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Expression Phase

以較深力度和較慢的吸嘔節奏刺激較大的乳量，大約每分鐘60次，讓母親可輕鬆餵哺母乳。
Follows with a deeper sucking and slower pumping rhythm with ~60 cycles/minute to maximize the milk flow and express milk gently and efficiently.

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Be our VIP and enjoy various welcome gifts and exclusive offers:

- Medela product samples and coupons
- Purchase discounts
- Gifts and exclusive offers from wide range of well-known brands
- Unlimited times to review webinars video
- Celki Point Reward - consume every \$1 at Celki VitalAire Centres or Medela Eshop can gain 1 Celki Point. Redeem \$1 discount with every 100 points collected

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Register and login to get your gifts NOW!
The welcome gift redemption coupons will be sent to your VIP account once your registration succeeds.
Register and login to get your gifts NOW!

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Stimulate, Build-up and Maintain Milk Production

- 2-Phase Expression breast pump
- Imitates the natural sucking pattern of babies. Starting with "Stimulation" mode to massage 2 minutes, followed by "Expression" mode to pump deeply and slowly, or switch between 2 modes anytime.
- 16 vacuum levels for mothers to reach maximum comfort
- Hygienic & Safe - "Closed System" keeps breast milk separate from the pump, preventing contaminations by bacteria or mold
- Proven to pump more breast milk in less time
- Widely used in hospitals
- For rent or purchase

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- 2 Phase Expression breast pump
- Intuitive multifunctional interface with:
 - Count-up timer
 - 10-sec delay start & pause functions
 - Pumping setting
 - Battery charge indicators
- Modern design with touch-screen
- USB cable port for powering up anywhere you are
- Compact design, weighs under 300 g (10 oz)
- Detachable lanyard loops round your wrist to keep breast pump secure
- Built-in rechargeable battery lasts up to 8 double pumping sessions
- USB port for powering up on the go
- Connects to free  Medela Family app to track pumping progress
- Medela PersonalFit Flex™ breast shields for your comfort and increased milk flow
- Closed system prevents breast milk from entering the tubing

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- 2段吸嘜模式奶泵
- 手柄頂端為刺激模式，底端為擠奶模式
- 配備PersonalFit Flex™ 舒悅版吸奶喇叭，提升舒適度，同時增加11.8%的泵奶量
- 105°角及橢圓形喇叭設計，可360度旋轉，更貼合乳房形狀
- 喇叭邊緣柔軟平滑，倍感舒適自然
- 方便操作及攜帶
- 2-Phase Expression breast pump
- Press handle upper end for stimulation while another end for expression
- PersonalFit Flex™ breast shield for increased comfort and 11.8% more milk
- 105° angle and oval shape breastshield can be rotated 360° are designed to better fit breast shape
- Soft rim for a comfortable, natural feeling
- Easy to operate, handy

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Safe to share breast pumps or use second-hand breast pumps?

Using or borrowing a used home-use grade breast pump is unhygienic and unsafe even when using new accessories. The system may allow the breastmilk to come into contact with and be retained in the pump. Shared use of the pump may lead to cross-infection among users. Only hospital grade pumps are designed in such a way that milk never touches the inner parts of the pump.

可置於手掌，非常小巧
Light and compact, can fit in the palm

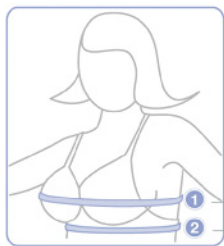
泵奶輔助用品 Related Product for Pumping

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步驟 Steps:

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Measure around the fullest part of the bustline, keeping the tape measure level.

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Measure around your body, just below your breasts.

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Check here for [size measurement chart](https://bit.ly/2twnPVR)

<https://bit.ly/2twnPVR>

母乳儲存及餵哺 Breastmilk Collection & Feeding

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Collection Bottle (150ml / 5oz)

(3個裝 3pcs/pack)

- 可直接連接美德牌奶泵及奶嘴
- Can be connected with Medela breast pump teat directly

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MRO0605

- Hygienically pre-sealed and disposable
- Double-walled bags with leak-proof double zipper, ensuring safe freezer storage
- Flat shape, quick to thaw breastmilk and space-saving
- With one transport pouch in each pack, keeping breastmilk bags organized and clean
- Made from food-grade, BPA-free material

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In general infant babies would need to feed 8 times a day.
To make feeding more conveniently, you are recommended to have 8 pieces of milk collection bottles.

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MR00522

- Curved bottom allows colostrum and breastmilk being drawn into a syringe easily
- 35ml; disposable
- Can be connected with Medela breast pump directly

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MR00516

- 80ml; disposable
- Time saving, improve efficiency
- Ideal for traveling or transferring breast milk to hospital
- Can be attached with Medela Calma Solitaire or teat
- Ready to use without prior cleaning, dispose after used

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MR00434

- Ideal for sanitizing collection bottles, breastshields and teats with microwave
- Chemical-free disinfection
- Eliminates 99.9% of common bacteria and germ
- Easy and quick: ready to use, works in minutes
- Convenient for working mothers to disinfect accessories at office
- Each pack has 5 bags, each bag can be used for 20 times

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After high-temperature sanitization, wait until the collection bottles cool down before taking them out, or the bottles may deform.

吸奶喇叭 Breastshield

PersonalFit Flex™ 舒悅版吸奶喇叭 Breastshield

細S	21mm	MR00662
中M	24mm	MR00663
大L	27mm	MR00664
加大XL	30mm	MR00665

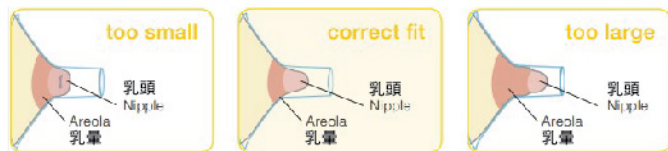
- 105°角及橢圓形設計，更貼合乳房形狀，減少乳房受壓程度
- 喇叭邊緣柔軟、平滑
- 喇叭可360度旋轉喇叭，以調整至最舒適的泵奶位置
- 平均每分鐘可多泵出11.8%奶量
- 需配合PersonalFit Flex™ 接駁器使用
- 105° angle and oval shape breastshield for better fit the shape of mom's breast
- Soft rim for a comfortable, natural feeling
- Breastshield can be rotated 360° to whatever position is most comfortable
- Removes 11.8% more milk per minute
- Used with PersonalFit Flex™ Connectors

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Refer to the images below while pumping in the expression phase (after the stimulation phase).

喇叭太細
乳頭摩擦管道側面。
Nipple rubs along the side of tunnel.

合適尺寸
乳頭置中且可自由移動。
Nipple is centered and moves freely.

喇叭太大
乳頭和過多乳暈被吸入管道中。
Nipple and excessive areola are pulled in tunnel.

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PersonalFit Flex™ Connector 接駁器

(每盒2個 2pcs/box)

- 適用於Freestyle Flex™, New Swing Maxi™, Solo™, Swing Flex™, Swing Maxi Flex™ 及 Freestyle™ 奶泵
- 封閉式系統設計，防止母乳流入喉管及摩打
- 讓媽媽可以較為後傾的角度泵奶，更加舒適
- 配合PersonalFit Flex™ 舒悅版吸奶喇叭及單泵喉管 / 雙泵喉管使用
- 組裝及清潔簡易
- For Freestyle Flex™, New Swing Maxi™, Solo™, Swing Flex™, Swing Maxi Flex™, Freestyle™
- With overflow protection, prevents milk overflow into the tubing and motor
- Allow to pump in a more reclined and comfortable positioning
- Used with PersonalFit Flex™ Breastshield and tubing
- Easy to handle and clean

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細S MR00672
中M MR00673
大L MR00674
加大XL MR00675

- For pregnant and breastfeeding mums
- Non-wired and seamless design
- Light, breathable fabric feels gentle and comfortable
- Super-soft and stretchy under-bust band
- Removable foam cups
- Drop-down cups with one-handed open and close clips for immediate skin-to-skin contact

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- Rehydrate and relief sore nipples
- 100% pure lanolin without any artificial additives or preservatives
- Can be used on baby's skin
- Purity surpasses USP standard, no need to remove before breastfeeding

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MR00209

- Collect breast milk while breastfeeding on the other side
- Soft silicone backs for comfortable fit

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MR00175

- Protect sore and cracked nipples, with vented fronts for faster healing
- Soft silicone backs for comfortable fit

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MR00214

- Prepare flat or inverted nipples for breastfeeding
- Soft silicone backs for comfortable fit
- Suggested to use during pregnancy from 32nd week onwards

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細S 16mm MR00460
中M 20mm MR00461
大L 24mm MR00462

- Protect sensitive, dry or cracked nipples during breastfeeding
- Enable mothers with flat or inverted nipples to breastfeed
- Open section for baby to stay in contact with mother
- Made from transparent, ultra-thin, soft silicone
- Safe and taste-free for baby

特殊情況餵奶用品 Special Needs Feeder

Medela 產品價目表 Price List

希柏奶瓶 Special Needs

- 免唇及裂顎嬰兒或吸嘍有困難的嬰兒使用
- For cleft lip, cleft palate babies or babies with nursing difficulties

MR00189

流量控制餵杯 SoftCup™ Feeder

- 可避免初生時乳頭混現象和控制量不會噏著
- Avoid nipple confusion and spillage with the special self-filling reservoir design

MR00236

餵杯 Baby Cup

- 需於醫護人員指導下使用
- 可避免乳頭混淆現象
- 邊口弧形設計方便餵哺
- For used under professional supervision
- Avoid nipple confusion
- Curve design for easy feeding

MR00261

歡迎下載Medela最新價目表
Download the latest price list

Follow us:

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www.medela.com.hk/home.aspx

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www.medelashop.com.hk

尚有更多產品，歡迎致電或Whatsapp查詢:
For full collection of products, please call or Whatsapp:

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2 吸管 Tubing

- 若母乳不慎回流至吸管内，請關機，拔下插頭，再以水清洗。詳情請參閱“使用說明書”。**吸管絕對不能以煮沸或用蒸汽消毒煲方式清潔。**
- 必須保持吸管乾爽。如有需要，請更換新的吸管。
- If milk accidentally gets into the tubing, turn off the pump and unplug from power source and clean with water. Please read the user guideline for details. **NEVER TRY TO PUT THE TUBING INTO THE BOILER OR STERILIZER FOR CLEANING.**
- Tubing must be completely dry. Replace as required.

Enjoy your
breastfeeding
journey